



E SEI IN MANI SICURE!

REFLEXX S.r.l.

INFORMATIVE NOTE

rev 00 del 01 marzo 2016

Product: reflexx 67 art. R67/M – art. R67/L – art. R67/XL – art. R67/XXL

Emessa da G.Isetti – Amm.re Unico

Pag.: 1/4

EXAMINATION GLOVES DISPOSABLE NITRILE

TRADEMARK REFLEXX 67strong

1. NAME AND ADDRESS OF MANUFACTURER:

REFLEXX S.R.L.

Via Passeri, 2

46019 VIADANA (MN)

Tel. 0375 833164 Fax. 0375 464504

2. GENERAL DESCRIPTION OF PRODUCT:

- disposable nitrile synthetic glove with five fingers, beaded cuff , ambidextrous , black color, textured surface.
- non-sterile gloves, powder free , internally chlorinated for easy donning and to reduce the risk of allergies.
- multi-purpose glove to protect the hand and / or the product and can be used for general purposes.
- glove indicated for mechanical uses.
- glove can be used as Personal Protective Equipment (PPE) also against chemicals and micro-organisms
- Unit For Sale in boxes of 100 gloves / pcs.
- Outer packagings containing n ° 10 packs.

Imballo		
Cod./Taglie	Confezione	Cartone
R67/M	8032891633045	8032891638040
R67/L	8032891633052	8032891638057
R67/XL	8032891633069	8032891638064
R67/XXL	8032891633076	8032891638071



E SEI IN MANI SICURE!

REFLEXX S.r.l.	INFORMATIVE NOTE	rev 00 del 01 marzo 2016
Product: reflexx 67 art. R67/M – art. R67/L – art. R67/XL – art. R67/XXL		
<i>Emessa da G.Isetti – Amm.re Unico</i>	Pag.: 2/4	

3. QUALITY AND COMPLIANCE WITH REGULATIONS:

- As a PPE category III risk according to IT Legislative Decree 475/92 implementing Dir 89/686/EEC (CIMAC, authorized Ministerial: 0465). The product is certified according to the EN 420 , EN 374 part I, II and III and EN 388.

4. DIMENSIONS AND PERFORMANCE:

Dimensions (in mm.)	size		Length (min.)	Width
	description size	size CE		
	M	8	240	90-99
	L	9	250	100-109
	XL	10	260	110-119
	XXL	11	270	120-129

Thickness (in mm.)	Palm (± 0.03)	0.11
---------------------	---------------------	------

EN 420	construction compliant	in compliance
	measures	in compliance
	donning	in compliance
	dexterity	minimum diameter of 5 mm peg collected

EN 374	test water loss	passed
	evidence of air leak	passed

EN 374-3		Performance level of permeation	Breakthrough time (permeation)
----------	--	---------------------------------	--------------------------------



E SEI IN MANI SICURE!

REFLEXX S.r.l.

INFORMATIVE NOTE

rev 00 del 01 marzo 2016

Product: reflexx 67 art. R67/M – art. R67/L – art. R67/XL – art. R67/XXL

Emessa da G.Isetti – Amm.re Unico

Pag.: 3/4

	Sulfuric acid 96 %	2	>30'
	Sodium hydroxide 40%	3	>60'
	diethylamine	2	>30'

EN 388		Protection Levels
	Resistance to abrasion, blade cut, tear , perforation	0
	Dexterity	5

EN 455-2:2009/A1	Tensile strength	≥6 N. (min)
------------------	------------------	-------------

For the certification of PPE category III, gloves reflexx 67 are tested with the following chemicals (EN 374): 96% sulfuric acid, sodium hydroxide 40%, diethylamine and are tested (legislation EN-388) for resistance to abrasion, to cutting by blade, for tearing and perforation.

Gloves reflexx 67 are controlled according to the sampling method ISO 2859 AQL 1.5 (level G1) for freedom from holes AQL 2.5 (level G1) for major defects and AQL 4.0 (level G1) for minor defects.

5. INSTRUCTIONS FOR USE AND STORAGE

1. Please read carefully the information note before using the glove
2. Please keep this information note for the duration of the useful life of the glove
3. Disposable nitrile glove - PPE : Risk Category III can also be used against chemicals and micro-organisms (Directive 89/686/EEC)
4. The product does not contain natural rubber latex
5. In rare cases, you may encounter transient skin reactions and hypersensitivity reactions. In case of a reaction discontinue use . Nitrile gloves Reflexx 67 are designed in order to eliminate allergic reactions associated with latex proteins . The absence of dust enables the use of the glove even to people who suffer from irritations caused by allergies to corn starch or a particularly sensitive skin
6. Avoid contact with aggressive substances such as fats , oils , acids, hydrocarbons, ketone solvents and other chemical agents



E SEI IN MANI SICURE!

REFLEXX S.r.l.

INFORMATIVE NOTE

rev 00 del 01 marzo 2016

Product: reflexx 67 art. R67/M – art. R67/L – art. R67/XL – art. R67/XXL

Emessa da G.Isetti – Amm.re Unico

Pag.: 4/4

7. Before using the product, test it in real conditions of use
8. The glove is a device subject to wear and tear so it is recommended the frequent replacement , and in all cases where visually you can see imperfections. Always wear gloves your hands are dry and clean Avoid exposure to sunlight and heat sources
9. Keep the packaging for further information and to ensure traceability
10. Keep the product in its original packaging in a dry place at a temperature not exceeding 40 ° C or below -10 ° C
11. Avoid exposure to sunlight and any heat sources
12. The expiry date is indicated on the packaging and it is valid when the product must is properly stored

REFLEXX Srl
Via Passeri, 2 - 46019 VIADANA (MN)
P.I. e C.F. 02085450209
Tel. 0375.833164 - Fax 0375.464504
e-mail: info@reflexx.com