



E SEI IN MANI SICURE!

REFLEXX S.r.l.

INFORMATIVE NOTE

rev 02 del 21 agosto 2015

Prodotto FEATHER LATEXX PF art. FLPF/S – art. FLPF/M – art. FLPF/L - art. FLPF/XL

Emessa da G.Isetti – Amm.re Unico

Pag.: 2/3

3. QUALITY AND COMPLIANCE WITH REGULATIONS:

- As a PPE category I risk according to IT Legislative Decree 475/92 implementing Dir 89/686/EEC. The product is certified according to the EN 420 and EN 374.

4. DIMENSIONS AND PERFORMANCE:

Dimensions (in mm.)	Size	Length (min.)	Width (+/-5)
	S	240	85
	M	240	95
	L	240	106
	XL	240	110

Thickness (in mm.)	Cuff (± 0.03)	0.07
	Palm (± 0.03)	0.09
	Fingers (± 0.03)	0.10

EN 420	construction compliant	in compliance
	measures	in compliance
	donning	in compliance
	dexterity	minimum diameter of 5 mm peg collected

5. INSTRUCTIONS FOR USE AND STORAGE:

1. Please read carefully the information note before using the glove.
2. Please keep this information note for the duration of the useful life of the glove.
3. Disposable Glove: PPE: Risk Category I (Directive 89/686 / EEC).



E SEI IN MANI SICURE!

REFLEXX S.r.l.

INFORMATIVE NOTE

rev 02 del 21 agosto 2015

Prodotto FEATHER LATEXX PF art. FLPF/S – art. FLPF/M – art. FLPF/L - art. FLPF/XL

Emessa da G.Isetti – Amm.re Unico

Pag.: 3/3

5. The glove is a device subject to wear and tear so it is recommended the frequent replacement , and in all cases where visually you can see imperfections. Always wear gloves your hands are dry and clean.

6.Avoid contact with aggressive substances such as fats , oils , acids, hydrocarbons and other chemical agents ; has low resistance to ketone solvents.

7.Before using the product, test it in real conditions of use

8.The product contains latex rubber. It may cause allergic reactions of type I hypersensitivity and delayed hypersensitivity type IV.

9.Keep the packaging for further information and to ensure traceability

10. Keep the product in its original packaging in a dry place at a temperature not exceeding 40° C or below -10 °C.

11.Avoid exposure to sunlight and heat sources.

12.The expiry date is indicated on the packaging and it is valid when the product must is properly stored.

REFLEXX Srl
Via Passeri, 2 - 46019 VIADANA (MN)
P.I. e C.F. 02085450209
Tel. 0375.833164 - Fax 0375.464504
e-mail: info@reflexx.com