



E SEI IN MANI SICURE!

REFLEXX S.r.l.	INFORMATIVE NOTE	rev 07 del 15 marzo 2015
Product FEATHER NITREXX		
art. FNX/ S – FNX /M – art. FNX /L - art FNX /XL		
<i>Emessa da G.Isetti - Presidente</i>		Pag.: 1/3

EXAMINATION GLOVES DISPOSABLE NITRILE POWDER FREE TRADEMARK FEATHER NITREXX

1. NAME AND ADDRESS OF MANUFACTURER:

REFLEXX S.R.L.
Via Passeri, 2
46019 VIADANA (MN)
Tel. 0375 833164 Fax. 0375 464504

2. GENERAL DESCRIPTION OF PRODUCT:

- disposable nitrile synthetic glove with five fingers, beaded cuff , ambidextrous , blue color.
- non-sterile gloves and powder free.
- multi-purpose glove to protect the hand and / or the product and can be used for general purposes.
- glove can be used as Personal Protective Equipment (PPE) against minimal risks.
- Unit For Sale in boxes of 100 gloves / pcs. in flat cardboard boxes recyclable
- Outer packagings containing n ° 10 packs made from recyclable corrugated cardboard

Imballo

			
Taglia	Articolo	Taglia	Cartone
FNX/S	 8008990232311	S	 8008990968166
FNX/M	 8008990232328	M	 8008990968159
FNX/L	 8008990232335	L	 8008990968142
FNX/XL	 8008990232397	XL	 8008990960641

3. QUALITY AND COMPLIANCE WITH REGULATIONS

As a PPE category I risk according to IT Legislative Decree 475/92 implementing Dir 89/686/EEC. The product is certified according to the EN 420 and EN 374



E SEI IN MANI SICURE!

REFLEXX S.r.l.	INFORMATIVE NOTE	rev 07 del 15 marzo 2015
Product FEATHER NITREXX		
art. FNX/ S – FNX /M – art. FNX /L - art FNX /XL		
<i>Emessa da G.Isetti - Presidente</i>		Pag.: 2/3

4. DIMENSIONS AND PERFORMANCE:

Dimensions (in mm.)	size		Length (min.)	Width
	description size	size CE		
	S	7	240	85
	M	8	240	95
	L	9	240	106
	XL	10	240	115

Thickness (in mm.)	Cuff (± 0.03)	0.04
	Palm (± 0.03)	0.06
	Fingers (± 0.03)	0.09

	measures	in compliance
	donning	in compliance
	dexterity	max dexterity class 5

5. INSTRUCTIONS FOR USE AND STORAGE

1. Please read carefully the information note before using the glove
2. Please keep this information note for the duration of the useful life of the glove
3. Disposable nitrile glove - PPE : Risk Category I against minimal risks (Directive 89/686/EEC)
4. The product does not contain natural rubber latex
5. In rare cases, you may encounter transient skin reactions and hypersensitivity reactions. In case of a reaction discontinue use . Nitrile gloves Feather Nitrex are designed in order to eliminate allergic reactions associated with latex proteins . The absence of dust enables the use of the glove even to people who suffer from irritations caused by allergies to corn starch or a particularly sensitive skin
6. Avoid contact with aggressive substances such as fats , oils , acids, hydrocarbons and other chemical agents
7. Before using the product, test it in real conditions of use
8. The glove is a device subject to wear and tear so it is recommended the frequent replacement , and in all cases where visually you can see imperfections. Always wear gloves your hands are dry and clean



E SEI IN MANI SICURE!

REFLEXX S.r.l.	INFORMATIVE NOTE	rev 07 del 15 marzo 2015
Product FEATHER NITREXX		
art. FNX/ S – FNX /M – art. FNX /L - art FNX /XL		
<i>Emessa da G.Isetti - Presidente</i>		Pag.: 3/3

9. Keep the packaging for further information and to ensure traceability
10. Keep the product in its original packaging in a dry place at a temperature not exceeding 40 ° C or below -10 ° C
11. Avoid exposure to sunlight and heat sources
12. The expiry date is indicated on the packaging and it is valid when the product must is properly stored
13. The product as PPE follows the procedures for the disposal of special waste if the use involves contact with regulated materials (chemical, radioactive and more). For general use the ppe follows the rules of waste.

REFLEXX Srl
Via Passeri, 2 - 46019 VIADANA (MN)
P.I. e C.F. 02085450209
Tel. 0375.833164 - Fax 0375.464504
e-mail: info@reflexx.com