reflexx®

IN SAFE HANDS

Reflexx S.p.A., Via Passeri 2, 46019 Viadana (MN) | ITALY

DISPOSABLE EXAMINATION NITRILE GLOVE

Reflexx N350
USER INFORMATION SHEET
rev. UIS 04 feb. 2022

CE 2797

PPE cat. III Reg. EU 2016/425 EN ISO 374-1:2016+A1:2018 TYPE B,
EN ISO 374-5:2016 VIRUS (ISO 16604:2004)
EN ISO 21420:2020



Chemical compound	EN ISO 374-1:2016+A1:2018 TYPE B	EN 374-4:2019		
OOD	Performance level	Breakthrough time	% degradation	
Sodium Hydroxide 40%	K	6	>480	2,8
Sodium Hypochlorite 10-13%	L	6	>480	29,9
Sulfuric Acid 50%	L	6	>480	50,8
Glycium bromide 5%	L	6	>480	>12,9
Formaldehyde 37%	T	3	>90	24,5
Glyoxyaldehyde 50%	L	6	>480	4,5
Phenol 0,1%	L	6	>480	9,4
n-Hexane	J	0	<10	45,7
Methanol in water 15%	L	6	>480	>12,3
Isopropanol 70%	L	0	<10	30,6
Nitric Acid 65%	M	0	<10	69,4
Acetic Acid 95%	N	0	<10	97,9
Ammonium Hydroxide 25%	O	0	<10	>0
Hydrogen Peroxide 30%	P	4	>120	32,1

Module D on going conformity by 2797 BSI Group The Netherlands B.V. - Say Building, John M. Keynesplein 9, 1066 EP Amsterdam, Netherlands
EU type examination (Module B) by 2777, SATRA TECHNOLOGY EUROPE LIMITED, Bracetown Business Park, Clonee, Dublin 15, D15 2YN2, Ireland.

OTHER COMPLIANCE:

- Medical Device CLASS I | Reg. EU 2017/745 - EN 455 1,2,3 & 4

Reg. CE 1935/2004 - Reg. UE 10/2011 - DM 21/03/1973
FOOD SAFE certified by: Chelab S.r.l., a Merieux NutriScience company Via Fratta, 25, 31023 RESANA (TV) Italy
EN 374-4: 2013 Degradation results indicate the change in puncture resistance of the gloves after exposure to the challenge chemical.

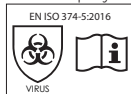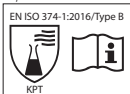EN

Declaration of Conformity on www.reflexx.com

■ Disposable examination nitrile glove ■ Powder free, non-sterile ■ Shape: ambidextrous; five fingers ■ Sizes: 6-10 ■ Please read this USER INFORMATION SHEET carefully before using the glove.

RECOMMENDATIONS OF USE

■ Multi-purpose glove to protect the hand and/or the handled product, can also be used for general uses. ■ This information does not reflect the actual duration of protection in the workplace and the differentiation between mixtures and pure chemicals. ■ The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only (except in cases where the glove is equal to or over 400 mm – where the cuff is tested also) and relates only to the chemical tested. It can be different if the chemical is used in a mixture. ■ It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion and degradation. ■ When used, protective gloves may provide less resistance to the dangerous chemical due to changes in physical properties. Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual use time significantly. For corrosive chemicals, degradation can be the most important factor to consider in selection of chemical resistant gloves. ■ Before usage, inspect the gloves for any defect or imperfections. ■ The penetration resistance has been assessed under laboratory conditions and relates only to the tested specimen. ■ Donning – Hold glove by the bead with one hand. Align the glove thumb with your other hand thumb and slide your hand into the glove, one finger into each glove finger. Pull by the glove palm to get a good fit. Don the other glove by the same procedure. ■ Doffing – Hold glove bead and pull toward the finger until the glove come off.

WARNINGS

■ Before using the product, try it in the real condition of use. ■ The glove is a device subject to wear, so frequent replacement is recommended and in all cases where imperfections are observed. ■ Wear gloves with dry, clean hands. ■ In rare cases, transient skin reactions and hypersensitivity reactions may occur. ■ In case of reaction, discontinue use and consult a doctor. ■ The user must inform the manufacturer and the relative authority of any incidents occurring during the use of the protective equipment.

STORAGE AND DISPOSAL

■ Store the gloves in their original packaging in a dry place. ■ Do not expose to direct sunlight and heat sources. ■ The expiry date is 3 years from the date of manufacture indicated on the sales package, and is valid for the product properly stored. ■ Gloves must be disposed of according to local regulations.

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