



 **INFORMATIVE NOTE**

Name: REFLEXX Care L600	
Date: 03/11/2017	
Rev: 00	

DESCRIPTION: Disposable glove non-sterile
Material: natural latex (protein content < 50 µg/g)
Shape: ambidextrous; five fingers, beaded cuff
Color: green
Minimum length: 240 mm
Thickness in the palm: 0.09 mm (+/- 0.03)
Finishing: externally textured surface; internally powder free, chlorinated and treated with aloe vera
Article/Sizes (CE EN 420): L600/XS (6) – L600/S (7) – L600/M (8) – L600/L (9) – L600/XL (10)
Packaging: 100 pieces / box; 10 boxes / carton
Tensile strength (455-2: 2009 / A1): ≥ 6 N



CONFORMITY AND PERFORMANCES

 Medical Device Class I Dir. 93/42/CEE – Dir 2007/47/CEE – EN 455 1,2,3, & 4
 Personal Protective Equipment (PPE) cat I only for minor risks – Dir. 89/686/EEC - EN420 – EN374

ISO 2859: AQL 1,5 (G1 level)

RECOMMENDATIONS OF USE

- Multi-purpose glove to protect the hand and/or the product, it can be used for general purposes
- Powder-free glove, also recommended for people allergic and/or sensitive to corn starch
- Glove treated with aloe vera, hydrating effect

INSTRUCTIONS FOR USE

- Before using the product, test it in real conditions of use
- Avoid contact with aggressive substances such as fats, oils, acids, hydrocarbons and other chemical agents
- The product contains latex rubber, that may cause allergic reactions of hypersensitivity (type I and IV)
- The glove is a device subject to wear and tear so it is recommended the frequent replacement, and in all cases where visually you can see imperfections.
- Wear gloves with dry clean hands
- Carefully read the Information Note before using the glove

STORAGE AND DISPOSAL

- Store the gloves in the original packaging in a dry place
- Avoid exposure to sunlight and heat sources
- Dispose the gloves in accordance with local regulations

