



E SEI IN MANI SICURE!

REFLEXX S.p.A.	INFORMATIVE NOTE	rev 010 del 26 ottobre 2017
Product REFLEXX 99 NHR art. R99/S- art. R99/M – art. R99/L- art. R99/XL -art. R99/XXL		
Issued by <i>G.Isetti – Amm.re Unico</i>		Pag.: 1/4

DISPOSABLE NITRILE GLOVES HIGH RESISTANCE

REFLEXX 99 NHR

1 NAME AND ADDRESS OF MANUFACTURER:

REFLEXX S.p.A.
 Via Passeri, 2
 46019 VIADANA (MN)
 Tel. 0375 758891 Fax. 0375 464504

2 GENERAL DESCRIPTION OF PRODUCT:

- ❑ disposable nitrile synthetic glove high strength
- ❑ glove with five fingers, ambidextrous, blue with long cuff, beaded cuff and Textured fingertips
- ❑ non-sterile gloves, powder free, internally chlorinated for easy donning and to reduce the risk of allergies.
- ❑ multi-purpose glove to protect the hand and / or the product and can be used for general purposes.
- ❑ glove used as a medical device by examination, even invasive , intended for temporary use
- ❑ glove can be used as Personal Protective Equipment (PPE) also against chemicals and micro-organisms
- ❑ Unit For Sale in boxes of 50 gloves / pcs.
- ❑ Outer packagings containing n° 10 packs



Imballo - Pack

50		10	
Taglia Size	Articolo Article	Taglia Size	Cartone Carton
R99 S	8032891632031	S	8032891637036
R99 M	8032891632048	M	8032891637043
R99 L	8032891632055	L	8032891637050
R99 XL	8032891632062	XL	8032891637067



E SEI IN MANI SICURE!

REFLEXX S.p.A.	INFORMATIVE NOTE	rev 010 del 26 ottobre 2017
Product REFLEXX 99 NHR art. R99/S- art. R99/M – art. R99/L- art. R99/XL -art. R99/XXL		
<i>Issued by G.Isetti – Amm.re Unico</i>		Pag.: 2/4

3 QUALITY AND COMPLIANCE WITH REGULATIONS

- ❑ As a PPE category III risk according to IT Legislative Decree 475/92 implementing Dir 89/686/EEC (CIMAC, authorized Ministerial: 0465). The product is certified according to the EN 420:2003 + A1 2009, EN 374-1:2016 TIPO B, EN 374-2:2015, EN 374-4:2013 e EN 374-5:2016
- ❑ As Medical Device Class I fall in between devices by invasive examination also intended for temporary use in accordance with Directive 93/42/EEC and subsequent Legislative Decree 37/2010 on the implementation Dir 2007/47/EEC . The product is in accordance with the EN 455 1-2-3 and 4. CND T01020204 – NON SURGICAL GLOVES NITRILE - PROGRESSIVE SYSTEM ATTRIBUTED TO DM: mis. S/1555790 M/1555791 L/1555792 XL/1555793 XXL/1563938.
- ❑ The product complies with ASTM-F-1671.
- ❑ The product may come into contact, for industrial use, with food as it complies with the relevant legislation: IT Directives 82/711/EEC, 85/572/EEC, 93/8/EEC, 97/48/EC, 2002/72/EC and subsequent amendments, regulations 1935/2004/EC, 1895/2005/CE, EU n.10/2011, DM 21/03/1973 and subsequent updates and changes, Presidential Decree 777/82 and subsequent updates and changes in accordance with the relevant regulations UNI EN 1186 1-15:2003. To such use carefully read the declaration of suitability for food contact (DICA).

4 DIMENSIONS AND PERFORMANCE

Dimensions (in mm.)	Size		Length (min.) (+/- 10)	Width
	description size	size CE		
	S	7	290	80-89
	M	8	290	90-99
	L	9	290	100-109
	XL	10	290	110-119
	XXL	11	290	120-129



E SEI IN MANI SICURE!

REFLEXX S.p.A.	INFORMATIVE NOTE	rev 010 del 26 ottobre 2017
Product REFLEXX 99 NHR art. R99/S- art. R99/M – art. R99/L- art. R99/XL -art. R99/XXL		
<i>Issued by G.Isetti – Amm.re Unico</i>		Pag.: 3/4

Thickness (mm)	Palm (± 0.03)	0.15
----------------	---------------------	------

EN 420:2003 + A1 2009	construction compliant	in compliance
	measures	in compliance
	donning	in compliance
	dexterity	minimum diameter of 5 mm peg collected

EN 374-2	test water loss	passed
	evidence of air leak	passed

EN 374-1:2016 TIPO B		Performance level of permeation	Breakthrough time (permeation)
	Sulfuric acid 96 %	3	>60'
	Sodium hydroxide 40%	3	>60'
	diethylamine	2	>30'
EN 374-4:2013	Degradation index		
	Sulfuric acid 96 %	93%	
	Sodium hydroxide 40%	47%	
	diethylamine	63%	

EN 455-2:2009/A1	Tensile strength	≥ 6 N. (min)
------------------	------------------	-------------------

For the certification of PPE category III, gloves reflexx 99 are tested for permeation and degradation with the following chemicals (EN 374-1:2016; EN 374-4:2013): 96% sulfuric acid, 40% sodium hydroxide, diethylamine. The gloves reflexx 99 are controlled according to the sampling method ISO 2859 AQL 1.5 (level G1) for absence from holes AQL 2.5 (level G1) for more defects and AQL 4.0 (level G1) for minor defects.



E SEI IN MANI SICURE!

REFLEXX S.p.A.

INFORMATIVE NOTE

rev 010 del 26 ottobre 2017

Product REFLEXX 99 NHR art. R99/S- art. R99/M – art. R99/L- art. R99/XL -art. R99/XXL

Issued by G.Isetti – Amm.re Unico

Pag.: 4/4

5. INSTRUCTIONS FOR USE AND STORAGE

1. It is recommended to read carefully the information note before the use of the glove
2. It is recommended to keep this information note for the duration of the useful life of the glove
3. The product does not contain natural rubber latex
4. In rare cases, you may encounter transient skin reactions and hypersensitivity reactions. In case of a reaction discontinue use . Nitrile gloves Reflexx 99 are designed in order to eliminate allergic reactions associated with latex proteins . The absence of dust enables the use of the glove even to people who suffer from irritations caused by allergies to corn starch or a particularly sensitive skin
5. Carefully read the times and levels of protection indicated for chemicals
6. Do not use gloves when there is a risk of getting caught in the moving parts of machinery
7. Before using the product, test it in real conditions of use
8. The glove is a device subject to wear and tear for which it is recommended that frequent replacement and in all cases in which you can see visually imperfections. Wear gloves always with clean, dry hands and after use accurately wash the hands
9. Keep the packaging for further information and to ensure traceability
10. Keep the product in its original packaging in a dry place
11. Avoid exposure to sunlight and heat sources
12. The expiry date is indicated on the packaging and it is valid when the product must is properly stored
13. Dispose of in accordance with applicable national, regional and local regulations

REFLEXX S.p.A.
Via Passeri, 2 - 46019 VIADANA (MN)
P.I. e C.F. 02085450209
Tel.: 0375/833164 - Fax: 0375/464504
e-mail: info@reflexx.com